

# RAPID ALCOHOL DISINFECTION

## Incidin™ Liquid

### ALCOHOL-BASED, READY-TO-USE SURFACE DISINFECTION

Incidin™ Liquid offers high reliability to the user through its broad spectrum of efficacy including Tuberculosis and Norovirus.

Incidin™ Liquid also provides:

- Short contact times
- Dye and fragrance free formulation
- Excellent material compatibility on alcohol resistant surfaces



# Incidin™ Liquid

## ALCOHOL-BASED, READY-TO-USE SURFACE DISINFECTION

### BROAD EFFICACY SPECTRUM IN SHORT CONTACT TIMES

Due to its broad efficacy spectrum in short contact times, Incidin Liquid is ideal even for intensive care area and other high risk areas. Contact times are detailed in table below.

### RELIABLE USE

Being ready-to-use and offering proven material compatibility on alcohol-resistant surfaces, Incidin™ Liquid ensures the reliable use for all application areas.

### DYE AND FRAGRANCE FREE

The dye and fragrance free formulation leaves no active residues and ensures a convenient product use.

### MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY UNDILUTED	CONTACT TIME
Bactericidal according to EN 13727 (30 sec.) /EN 16615 (1min)	1 min**
Yeasticidal according to EN 13624 (30 sec.) /EN 16615 (1min)	1 min**
Tuberculocidal according to EN 14348 /EN 16615 (method)	1 min**
Effective against enveloped viruses according to EN 14476	30 sec**
Limited spectrum virucidal according to EN 14476 (Adenovirus - 1min, Norovirus - 30 sec.)	1 min**

\* dirty conditions \*\* clean conditions

### PACKAGING AND ORDERING INFORMATION

PRODUCT	PACKAGING UNIT	ORDER CODE
Incidin™ Liquid Bottle	12 x 1 L	3096220
Trigger Spray Head	50	10005074

### MEDICAL DEVICE AND BIOCIDES USE SAFELY

Please read label and product information.

### APPLICATION AREA



**Medical Device:** Ready-to-use disinfection liquid for non-invasive medical devices.

**Biocide:** For general surfaces.

### INSTRUCTIONS FOR USE

#### 1L with sprayer:

Clean the surface upfront. Spray surface from approx. 30cm, wipe the surface with a clean wipe and leave to dry. Alternatively, apply the product to a clean wipe, wipe the surface completely and leave to dry. Respect the indicated contact time for the required antimicrobial activity.

Medical Device: Rinsing is unnecessary unless when areas to be treated are intended to come in contact with skin or mucosa. Please consult manufacturer's instruction for use for cleaning and disinfection of medical devices. Check compatibility of the alcohol sensitive surfaces before use.

### INGREDIENTS

100g contains: 35g Propan-2-ol (CAS N° 67-63-0), 25g Propan-1-ol (CAS N° 71-23-8).

### STORAGE

Product should be stored in the original container between 0°C and 25°C in a cool, dry location out of direct sunlight.



FOR PROFESSIONAL USE ONLY